

Instruction Manual

Bath Board




BB01WHI

SectorCare

<https://sectorcare.com.au>



Product Overview

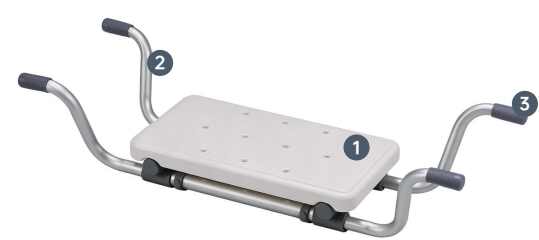

- 1 Seat Pad
- 2 Main Frame
- 3 Anti-slip Tips

Assembly Instructions



Base ×1



Seat Pan ×1



Armrest Tubes ×4



Curved Gasket ×2



U-shaped Clips ×2



Screws ×2



Instruction Manual ×1

Product Installation

1. Remove the product from the packaging.



2. Flip the Seat Pan upside down, lay the Base flat into the recess, then fit and fasten securely with Screws and Curved Gaskets, as illustrated.



3. Align the four Armrest Tubes with the fittings on both sides of the Base and slot them in place, then secure using U-shaped Clips, as illustrated.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.