

Instruction Manual

Aluminum Commode Chair



CMAL01MSIL

SectorCare

Contents

Safety Instructions	1
Product Installation	2-4

Safety Instructions

Please read this user manual carefully before using the commode. Keep this manual for future reference. If you do not fully understand the instructions, warnings, or precautions in this manual, consult your dealer or a qualified technician before using the product. Improper use or incorrect installation may result in injury.

When using the product, basic safety precautions should always be followed:

- 1 Consult your physician or healthcare professional before use to ensure suitability.
- 2 Use the product only on a flat, stable surface.
- 3 Ensure all legs are evenly adjusted and securely locked before use.
- 4 Make sure all components are properly installed and tightened.
- 5 Ensure the commode bucket is correctly positioned before use.
- 6 DO NOT exceed the maximum weight capacity.
- 7 Distribute body weight evenly when sitting.
- 8 Inspect the product regularly for damage or wear. Replace if necessary.
- 9 Keep away from open flames or high heat sources.
- 10 DO NOT hang items on the frame, as this may affect stability.
- 11 DO NOT stand on the commode chair.
- 12 Use extra caution on wet or slippery surfaces.
- 13 DO NOT use on stairs or uneven surfaces.
- 14 DO NOT use on escalators.
- 15 DO NOT leave or store the product outdoors.



Product Installation


1. Remove all products and accessories from the package.




2. Attach the backrest to the seat. Turn the seat upside down, align the holes, and secure the backrest using the screws provided. Tighten all screws firmly with the appropriate tools.




3. Insert the main leg frame into the seat slots on both the left and right sides (as shown in Step 1). Secure the frame using screws with washers and tighten firmly (as shown in Step 2). Attach the anti-slip tips to the legs (as shown in Step 3).

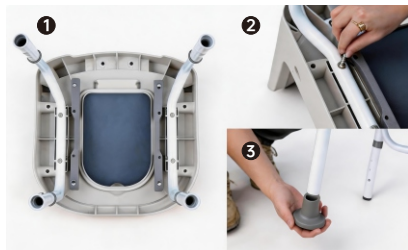

4. Attach the short horizontal tube to the front legs. Insert the screws with washers through the tube and secure them with nuts. Tighten firmly to ensure stability.

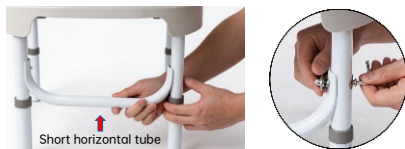

5. Attach the long horizontal tube to the back legs. Insert the screws with washers through the tube and secure them with nuts. Tighten firmly to ensure stability.

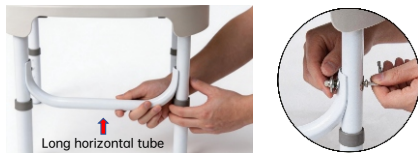

6. Align the holes and press the button on the armrest. Insert the armrest into the frame while holding the button, then release to lock it securely in place.



7. Align the holes and press the button on the armrest. Insert the armrest into the frame while holding the button, then release to lock it securely in place.



8. Place the commode bucket into the seat opening. Ensure it is securely positioned before use.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>