

Instruction Manual

Aluminium Commode Chair



CMAL02SSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



1 Backrest

2 Aluminium Frame

3 Armrest

4 Seat

5 Bedpan

6 Adjustable Leg
Tube

Product Installation

Parts List:



Main Frame*1



Seat Board*1



Bedpan*1




Foot Tube*4



Backboard *1

1. Take out all products and accessories after opening the package.

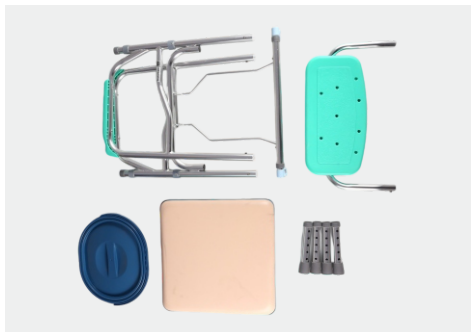

2. Unfold the main frame, then press the blue buckle to lock it onto the pole as shown in the figure.



3. Take out 4 leg tubes, align them with the joints, insert and secure as shown in the figure.



4. Lay the seat board flat on the main frame, press the bottom buckles to fix it to the poles as shown in the figure.



5. Align the backrest board with the connectors beside the armrests, press down to insert and secure, as shown in the figure.



6. Hang the bedpan flat on the bottom poles of the main frame as shown in the figure.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.