

Instruction Manual

Aluminum Commode Chair



CMAL02SSIL

SectorCare

Contents

Safety Instruction	1
Product Overview	2
Product Installation	3-5

Safety Instructions

When using the product, basic safety precautions should always be followed:

- 1 Ensure all parts are properly assembled and securely tightened before use.
- 2 Place the chair on a flat, stable surface before sitting.
- 3 DO NOT exceed the maximum weight capacity.
- 4 Ensure the rubber tips are intact and not worn or damaged. Replace if necessary.
- 5 DO NOT use the chair on uneven or slippery surfaces.
- 6 Sit down slowly and evenly to maintain balance.
- 7 DO NOT stand on the chair.
- 8 Ensure the pail is properly positioned before use.
- 9 Clean the chair regularly to maintain hygiene.
- 10 Keep the chair in a dry environment when not in use.



Product Overview

Congratulations on your purchase of this aluminum commode chair. This product is manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

This commode chair is a multi-functional chair, can be used as commode chair, bath chair and wheelchair. And it is one button to fold, which is easy for operating and space saving.

Elements of the Commode Chair



1 Backrest

2 Aluminum Frame

3 Armrest

4 Front Cross
Brace

5 Bedpan

6 Adjustable Leg
Tube

Product Installation

Assembly Instructions

Remove the commode chair and all accessories from the carton. Turn the armrests back on both sides to unfold the seat, then fully open the chair.



1. Pull the frame outward to unfold the chair, then secure it using the locking clasp.



2. Attach the buckle to both sides of the frame and secure it tightly with screws.



3. Remove the protective tube plugs, then install the backrest.



4. Insert the bedpan and place the seat pad securely onto the frame.



7. Adjust the armrests to an appropriate height. Align the holes, then insert the M8 screws and tighten securely.



8. Adjust the backrest to an appropriate height. Align the holes, then insert the M8 screws and tighten securely.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>