

Instruction Manual

Folding Aluminium Walker 3 in 1 Function



CSW01MSIL

SectorCare

Product Overview

Congratulations on your purchase of our products. Our walkers are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



1 Hand Grip

2 Push-Button
Folding Mechanism

3 Commode seat

4 Front Cross
Brace


5 Fixed/ Reciprocal
Mechanism

6 Height Adjustable
Leg

Assembly Instructions



Main Frame x1

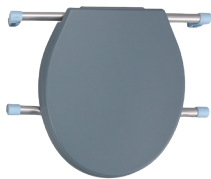

Toilet Seat x1



Commode Bucket x1



Hex Screws x2



Hex Nuts x2



Instruction Manual x1

Product Installation

- 1. Take out the product and accessories.



2. Unfold the Main Frame, as illustrated:



3. Place the Toilet Seat flat on the main frame, press down the blue clips to secure, as illustrated:



4. Take out the Hex Screws and Hex Nuts. As illustrated, fit them onto the blue clips and twist to tighten and secure.



5. Lift the lid of Toilet Seat, place Commode Bucket onto the frame.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>