

Instruction Manual

Folding Aluminum Walker 3 in 1 function



CSW02MSIL

SectorCare

Contents

Safety Instructions	1
Product Overview	2
Product Installation	3
Product Guidelines	3-4

Safety Instructions

When using the product, basic safety precautions should always be followed:



- ① Each individual should ALWAYS consult with your physician or therapist to determine proper adjustment and usage.
- ② ALWAYS use caution when using the walker, especially on wet or icy surfaces.
- ③ Make sure that the accessories are properly installed and securely in place before use.
- ④ Ensure that the rubber tips and/or plastic glide tips are not ripped, worn or missing. Replace the tips as necessary before using walker. DO NOT use rubber tips on rear leg extensions when using wheels on the front legs.
- ⑤ Make sure that the user's weight is distributed evenly and directly over the walker.
- ⑥ The walker is not intended to support the full weight of the user.
- ⑦ Check that all labels are present and legible. Replace if necessary.
- ⑧ Keep the walker away from open flames. It may cause injury to the user and damage the walker.



Product Overview

Congratulations on your purchase of our products. Our walker are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



- | | | |
|----------------------------|--|--------------------------------|
| 1 Hand Grip | 2 Push-Button Folding Mechanism | 3 Commode Seat |
| 4 Front Cross Brace | 5 Fixed/ Reciprocal Mechanism | 6 Height Adjustable Leg |

Product Installation

1. Remove the walker and all accessories from the carton.
2. Swing both side frames outward from the front cross brace and unfold the frame completely until an audible "click" is heard.
3. Press the snap button, extend or retract the leg to the raise or lower the height.
4. Adjust the leg extension to the desired height and allow the snap button to snap into the height adjustment hole. (An audible "click" will be heard)
5. Make sure that the walker is level.

Product Guidelines

Use as a Walking Aid (Fixed Mode)



Use as a Walking Aid (Reciprocal Mode)



Use as a Standing Aid



Use as a Toilet Safety Rail



Use as a Commode Chair



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>