

Instruction Manual

Wheelchair Lift



EWCLF1

SectorCare

Usage Operation

Instructions

1. Remove the lift packaging.
2. Check whether the wheels are loose. If any wheels are loose, tighten the screws using the appropriate tools.
3. Push to the usage site, lock the 4 wheels and brakes.
4. Place the wheelchair on the platform and fasten the safety belt.



5. Use the remote control by pressing the "UP" or "DOWN" button to raise or lower the lift platform.



6. Raise to the appropriate height, open the seat belt and remove the wheelchair.



7. Lower the lift to the bottom, retract the auxiliary wheels, and complete the operation.

Precautions.

1. Do not put your hand here during the lift operation.



2. Please step down the wheel lock before operation.
3. Release the wheel lock when moving the lift Rated load capacity $\leq 40\text{KG}$.

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>