

Instruction Manual

Folding Steel Bedrail



FBRST01WHI

SectorCare

Contents

Safety Instructions	1
Product Overview	2
Product Installation	3

Safety Instructions

Please read this user manual carefully before using the Bed Assist Rail. Keep this manual for future reference. If you do not fully understand the instructions, warnings, or precautions in this manual, consult your dealer or a qualified technician before using the product. Improper use or incorrect installation may result in injury.

When using the product, basic safety precautions should always be followed:

- 1 Check each part carefully before use. If the wearing parts are abnormal, please replace them in time.
- 2 Check the product regularly every week.
- 3 This style can be installed with a mattress/no mattress. If the thickness of the mattress is more than 7cm, it can be directly put under the mattress and fixed with a seat belt.
- 4 If there is no thick mattress, it can be screwed to the bed board. The main material is flat tube, which is more comfortable.
- 5 The surface of the bed rail is powder coated. The surface is smooth, delicate and durable. With 6 meters long seat belts for beds of different widths.
- 6 Applicable people: elderly people, patients, pregnant women, etc. who are inconvenient to get up.
- 7 This product should be placed in a dry, ventilated, stable, non-corrosive indoors.
- 8 The size of the product in the manual is manually measured, and there is an inaccuracy of 1~5cm. Please understand that.



Product Overview

Congratulations on your purchase of our products. Our Bed Assist Rail are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

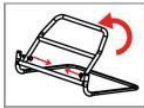
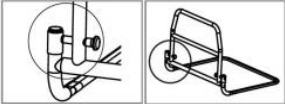
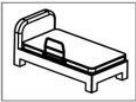
Main Frame Parts



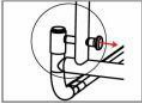
- 1 Storage Bag
- 2 Foam Handle
- 3 Folding Mechanism
- 4 Steel Frame

Product Installation

Installation Procedure

1 Open the package, take the product out.	
2 Pull the bolt on both sides and unfold the bed rail.	
3 Both sides of the bolt must be stuck in the hole before use. Pass the seat belt through the bed and the bed rail, tie the rail to the bed, and adjust the length of the seat belt.	
4 Position the bed rail on the bed and secure it using a Phillips screwdriver.	
5 Place the mattress over the base frame to complete installation.	

Folding Procedure

1 To fold the bed rail, pull the bolt on both sides.	
2 Rotate the bed rail outward simultaneously to fold it down.	
3 The bed rail is now in the folded position.	

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>