

Instruction Manual

Folding Aluminum Walker with seat



FWALSEATMSIL

SectorCare

Contents

Safety Instructions	1
Product Overview	2-3
Product Installation	3-4
Product Guidelines	4-5

Safety Instructions

Please read this user manual carefully before using the walker with a seat. Keep this manual for future reference. If you do not fully understand the instructions, warnings, or precautions in this manual, consult your dealer or a qualified technician before using the product. Improper use or incorrect installation may result in injury.



When using the product, basic safety precautions should always be followed:



- ① Ensure the walker is fully opened and locked before use.
- ② Use only on flat and stable surfaces.
- ③ All rubber tips must remain in contact with the ground during use.
- ④ Ensure toilet bucket is securely fitted before commode use.
- ⑤ Do not exceed maximum weight capacity.
- ⑥ Keep both hands on handles when walking.
- ⑦ Do not use on stairs or slippery surfaces.
- ⑧ Regularly inspect all fittings before use.



Product Overview

Congratulations on your purchase of our products. Our walker are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



- 1 Hand Grip
- 2 Push-Button Folding Mechanism
- 3 Main Frame
- 4 Front Cross Brace
- 5 Push-Button Folding Mechanism
- 6 Height Adjustable leg

Seat

Seat X1



Nut X2



Product Installation

1. Take out the walker and all of the accessories from the carton.
2. Open the walker as shown in Pic 1.
3. Pay attention to rotating the buckle in the correct direction as Pic 2.
4. Place the seat onto the tubes for both sides and press down as Pic 3.

Pic 1



Pic 2



Pic 3



Ensure the long buckle is positioned at the front, then tighten the screw. Repeat the same steps on the other side.



Product Guidelines

Using the Seat

Flip-up the seat and secure it tightly with velcro tape, making it easy to use for both walking and resting.



Standard Walking Method

Lift the walker and take a step forward, then the user moves one step forward.

Reciprocal Walking Method

Lift up the left side and move forward one step, then user go one step forward. Lift up the right side lifts forward and takes a step forward.

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>