

Instruction Manual

Folding Aluminum Walker



FWALSSIL

SectorCare

Contents

Safety Instruction	1
Product Overview	2
Product Installation	3
Product Guidelines	4

Safety Instructions

When using the product, basic safety precautions should always be followed:

- 1 Each individual should always consult a physician or therapist to determine proper adjustment and usage.
- 2 Always use caution when using the walker, especially on wet or slippery surfaces.
- 3 If the walker is exposed to extreme temperatures (above 37°C or below 0°C), high humidity, or becomes wet, ensure all parts are dry before use.
- 4 Ensure all accessories are properly installed and securely fastened before use.
- 5 Ensure rubber tips and/or plastic glide tips are not worn, cracked, or missing. Replace if necessary before use.
- 6 Make sure the user's weight is distributed evenly and directly over the walker.
- 7 The walker is not intended to support full body weight continuously.
- 8 Ensure all labels are present and legible. Replace if necessary.
- 9 Keep away from open flames or heat sources.
- 10 Do not hang items on the side frames or folding mechanisms, as this may affect stability.
- 11 Do not sit or lean on the crossbar.
- 12 Use extra caution when walking on uneven or sloping surfaces.
- 13 Do not use the walker on stairs.
- 14 Do not use the walker on escalators.
- 15 Do not leave or store the walker outdoors.



Product Overview

Congratulations on your purchase of our products. Our walker are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts Description



- | | | |
|----------------------------|--|--------------------------------|
| 1 Hand Grip | 2 Push-Button Folding Mechanism | 3 Main Frame |
| 4 Front Cross Brace | 5 Fixed/Reciprocal Mechanism | 6 Height Adjustable leg |

Product Installation

1. Take out the walker and all of the accessories from the carton.
2. Swing both side frames outward from the front cross brace and unfold frame completely until an audible "click" is heard.

Setting & Adjusting the Walker

- Press the snap button, extend or retract the leg to the raise or lower the height.
- Adjust the leg extension to the desired height and allow the snap button to snap into the height adjustment hole. (An audible "click" will be heard)
- Make sure that the walker is level.

Fixed / Reciprocal Adjustment

- Press the folding button and fold the side frames slightly.
- Fixed movement mode: press the snap button into upper hole (An audible "click" will be heard). The walker moves forward as a single unit.
- Reciprocal movement mode: press the snap button into lower hole (An audible "click" will be heard). The walker moves one side at a time, in step with the user.

Product Guidelines

Standard Walking Method

Lift the walker and take a step forward, then the user move one step forward.



Reciprocal Walking Method

Lift up the left side and move forward one step, then user go one step forward. Lift up the right side lifts forward and takes a step forward.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>