

Instruction Manual

Folding Aluminum Walker with wheels and brakes



FWALWBMSIL

SectorCare

Contents

Safety Instruction	1
Product Overview	2
Product Installation	3

Safety Instructions

Please read this user manual carefully before using the rollator walker. Keep this manual for future reference. If you do not fully understand the instructions, warnings, or precautions in this manual, consult your dealer or a qualified technician before using the product. Improper use or incorrect installation may result in injury.

When using the product, basic safety precautions should always be followed:

- 1 Always consult a physician or therapist to determine proper adjustment and use.
- 2 Use the rollator only on flat, stable surfaces.
- 3 Always apply the brakes when standing still or before sitting (if equipped with a seat).
- 4 Ensure both brakes are functioning properly before use.
- 5 Keep both hands on the handles while walking.
- 6 Do not use the rollator on stairs or escalators.
- 7 Use extra caution on slopes, ramps, or uneven surfaces.
- 8 Do not exceed the maximum weight capacity.
- 9 Ensure all parts are properly assembled and securely fastened before use.
- 10 Regularly check for wear, loose components, or damage.
- 11 Do not hang heavy items on the frame, as this may affect stability.
- 12 Keep away from open flames or extreme heat.
- 13 Do not leave or store the rollator outdoors.



Product Overview

Congratulations on your purchase of our products. Our walker are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Characteristics

This front-wheel walker with brakes features a reinforced aluminium frame for stability and durability. It is equipped with 5" front wheels and ergonomic hand brakes with a parking lock function. The rear legs are fitted with anti-slip rubber tips for added stability.

The walker provides reliable support and comfort during use. The height can be easily adjusted to suit the user. When folded, it is compact and convenient for storage and transport.

Elements of the walker



1 Hand Grip

2 Frame Folding Button

3 Height adjustment leg

4 5" Front Wheels

5 Anti-slip Rubber Tips

Product Installation

Unfolding and Folding

- To unfold the walker, pull the side frames outward until the frame is fully opened and locked into position.
- To fold the walker, press the folding buttons located at the top of the frame and fold the side frames inward.

Height Adjustment

To adjust the leg height, press the metal pin and slide the leg to the desired height. Ensure the pin locks securely into the adjustment hole. Adjust all four legs evenly to maintain balance.

Note! Ensure all legs are set at the same height before use.

Function Selection

The walker can be configured with wheels at the front and rubber tips at the rear for stability. The hand brakes include a parking lock function:

- To lock the brakes, push the brake handles downward until they click into place.
- To release the brakes, squeeze the brake handles upward until the lock disengages.

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>