

Instruction Manual

Folding Aluminum Walker with wheels



FWALWMSIL

SectorCare

Contents

Safety Instruction	1
Product Overview	2
Product Installation	3-4

Safety Instruction

When using the product, basic safety precautions should always be followed:

- 1 Ensure all parts are properly assembled and securely tightened before use.
- 2 Use the walker on a flat, stable surface.
- 3 DO NOT exceed the maximum weight capacity.
- 4 Ensure the rubber tips are intact and not worn or damaged. Replace if necessary.
- 5 Do not use the walker on uneven or slippery surfaces.
- 6 Use caution when walking and move slowly to maintain balance.
- 7 Do not use the walker on stairs or escalators
- 8 Ensure the height adjustment buttons are securely locked before use.
- 9 Keep the walker clean to maintain performance and hygiene.
- 10 Store the walker in a dry environment when not in use.



Product Overview

Congratulations on your purchase of our products. Our walker are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts Description



1 Hand Grip

2 Push-Button
Folding Mechanism

3 Main Frame

4 Height Adjustment
leg

5 Rear Rubber Tip

6 Front Wheel

Product Installation

Setting the Walker

1. Remove the walker from the carton.
2. Swing both side frames outward, away from the front cross brace until an audible "click" is heard.



Folding the Walker

1. Press the "push" button on each side and fold the side frames inward toward the front cross brace.



Installing / Adjusting Leg Extensions

1. Press the snap button and slide the leg extension onto the walker leg.
2. Adjust the leg extension to the desired height and allow the snap button to snap into the height adjustment hole. (An audible "click" will be heard)
3. Ensure that the walker is level.



WARNING:

The walker **MUST** be level before use. When using wheeled attachments on the front legs, the rear leg extensions **MUST** be adjusted to keep the side frames level.

Pinch points may occur during installation and adjustment. Keep hands and body clear of all pinch points.

Wheeled leg extensions **MUST** be installed with the wheels positioned on the outside of the walker.

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>