

Instruction Manual

Transfer Bath Bench with Small Backrest



SBALO1SSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



- 01 Backrest
- 02 Armrest Pipe
- 03 Seat Board 1
- 04 Seat Board 2
- 05 C-shape Latches
- 06 Footrest Pipes

Assembly Instructions



Dual Seat Board x1



Single Seat Board x1



Backrest x1



Round Armrest Tubes x1



Large Suction Foot Tubes x2



Small Suction Foot Tubes x2



U-Shaped Clips x4



Instruction Manual x1

Product Installation


1. Remove the product from the packaging.



2. Align the Single Seat Board and Dual Seat Board with the joining fittings, slot them together and secure firmly, as illustrated.




3. Insert each of the four Foot Tubes into the matching fittings on the base, then secure them in place with U-Shaped Clips, as illustrated.

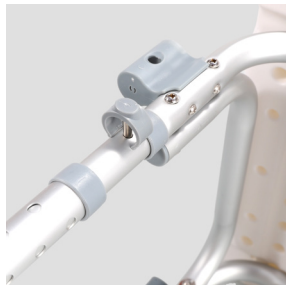

4. Align the Round Armrest Tubes to the side connection ports of the pre-assembled seat board and slot them securely into position, as illustrated.



5. Fit the Backrest to the connectors of the pre-assembled seat board and slot it securely into position. Assembly complete, as illustrated.

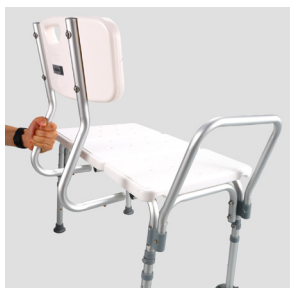

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.