

# Instruction Manual

Transfer Bath Bench with small back Heavy Duty




**SBAL02HDSSIL**

**SectorCare**

# Usage Operation

---

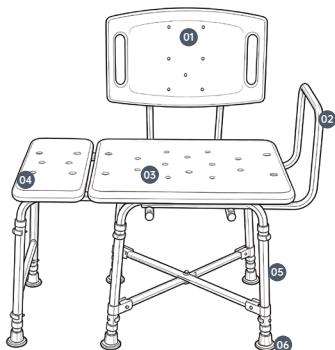

**01** Backrest

**02** Armrest

**03** Seat Board

**04** Extension Seat Board

**05** Main Base Frame

**06** Non-slip Footrest Pad

## Assembly Instructions

1. Remove the product from the packaging.




Square Seat Panel\* 1



Rectangular Seat Panel\* 1




Backrest Panel\* 1

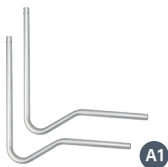

Backrest Tube\* 2



Backrest Fixing Tube\* 2

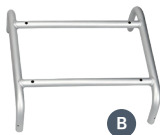

Main Frame\* 1



Adjustable Leg Tube\* 6



Foot Tube\* 4



Reinforcement Tube\* 2



Curved H-shaped Frame\* 1



Straight H-shaped Frame\* 1



Armrest\* 1



Long Screw\* 4



Standard Screw\* 4



Short Screw\* 13



Nut\* 5



Large Curved Gasket\* 14



Small Curved Washer\* 4



Round Washer\* 8



Silicone Washer\* 8



U-shaped Clip\* 12



Instruction Manual\* 1

## Backrest Installation

1. Place the Backrest Tube (A1) into the recess of the back panel, then fasten the four connection points respectively with Standard Screws and Small Curved Washers.



2. Insert the Backrest Fixing Tube (A2) into the corresponding connectors as illustrated.




## Foot Frame Installation

1. Take out the Main frame (B) and Foot Tubes (B2). Insert the four foot tubes into their matching connection ports and secure them with U-shaped Clips as illustrated.



2. Take out the four Adjustable Leg Tubes (B1), align each with the connector of the corresponding Foot Tube (B2) and insert them, then secure with U-shaped Clips as illustrated.

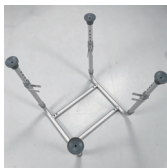

3. Retrieve the Reinforcement Tube (B3). Arrange it in a cross formation, align with the connectors and fasten securely using Standard Screws, Nuts and Large Curved Gaskets.

Next, fit the cross assembly into the matching ports of the Foot Tubes, then tighten each joint with Standard Screws, Round Washers and Nuts respectively.



## Small Chair Assembly

1. Take out the Main frame (B) and Foot Tubes (B2). Insert the four foot tubes into their matching connection ports and secure them with U-shaped Clips as illustrated.



2. Take out the Straight H-shaped Frame (C1). Align it with the connectors of the Curved H-shaped Frame (C) and lock in place with U-shaped Clips.

Then retrieve the two Adjustable Leg Tubes (B1). Insert each into the corresponding connectors on the Straight H-shaped Frame (C1) and secure with U-shaped Clips.




## Overall Assembly

1. Take out the Square Seat Panel. Position the pre-assembled leg frame into the recess, then fit a Round Washer onto each connector of the leg frame. Lay the Armrest (D) flat in alignment with the Round Washers, and place a Silicone Washer at each connector point of the Armrest as illustrated.



2. Align the pre-assembled backrest above the connectors of the Armrest (D), place a Large Curved Gasket at each connection point, then tighten using Long Screws through the connectors as illustrated.

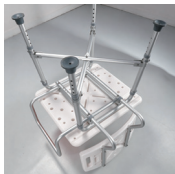

3. Finally, join all pre-assembled chair components together. Fit Large Curved Gaskets at each joint, align the connectors and tighten with Standard Screws. Assembly is now complete.



## LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

# SectorCare



<https://sectorcare.com.au>