

Instruction Manual

Foldable Shower Bench With Wall Mount



SCAL02MSIL

SectorCare

Contents

Safety Instructions	1
Product Overview	2
Product Installation	3-5

Safety Instructions

The shower chair **MUST** be level before use. Pinch points may occur when folding or adjusting the legs. Keep hands and body clear of moving parts.

CARE AND MAINTENANCE

Maintenance should be done at least every six months.

DO NOT use abrasive detergents, cleaners, or rough cloths.

- 1 Check regularly to ensure all mounting brackets and legs are securely fixed.
- 2 Use a mild detergent and warm water to clean.
- 3 Dry with a soft cloth to avoid moisture build-up.
- 4 Inspect and tighten all screws and bolts periodically.
- 5 Ensure both legs are adjusted to the same height so the seat remains level.
- 6 Make sure leg buttons are fully locked into position before use.
- 7 Inspect before each use.
- 8 Use only on a stable and level surface.
- 9 **DO NOT** stand on shower chair.



Product Overview

Congratulations on your purchase of our products. Wall mounted shower chair are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



- 1 Seat
- 2 Wall Mounting Bracket
- 3 Anti-slip Feet
- 4 Height Adjustable Legs

Package Contents

Seat X1



Wall Mounting Bracket X2



Anti-slip Tips X2



Wall Mounting Screws X8



Product Installation

Press the seat board with one hand and pull the feet with the other hand to unfold the product.




Firstly, please adjust the installation position and the appropriate height before fixing the brackets.



Use a marker to mark the drilling position.



Drill holes (drilling bit diameter 8mm, hole depth not less than 5mm).

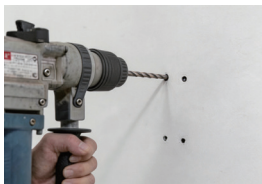

Insert all screws into the fixing plate.



Gently tap the screw inside the holes with a hammer.



Use a wrench to tighten the screws.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>