

# Instruction Manual

## Aluminum Bath Stool



**SSAL01MSIL**

**SectorCare**

# Contents

---

|                      |   |
|----------------------|---|
| Safety Instructions  | 1 |
| Product Overview     | 2 |
| Product Installation | 3 |
| Product Guidelines   | 4 |

# Safety Instructions

**When using the product, basic safety precautions should always be followed:**

- ❶ DO NOT exceed the maximum user weight limit.
- ❷ DO NOT attach any parts or accessories from other manufacturers to this product.
- ❸ DO NOT use if the rubber feet are damaged.

## CARE AND MAINTENANCE

- ❶ Inspect the product regularly for signs of wear or damage.
- ❷ Clean the shower chair regularly using a damp cloth with mild detergent. Dry all surfaces thoroughly before use.
- ❸ Check all components, including screws, backrest, rubber tips, and tubing, for damage or looseness. Replace the product if any part is damaged.



# Product Overview

---

Congratulations on your purchase of our products. Our shower stool are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

## Main Frame Parts




- 1 Seat
- 2 Height Adjustable Legs
- 3 Main Frame
- 4 Anti-slip Tips

# Product Installation

---

1. Carefully remove all packaging. Check the product thoroughly for damage or obvious faults.
2. Place the seat plate upside down on a flat surface. Align the main frame with the pre-drilled holes and secure it using the provided screws and washers. Tighten firmly as shown in the image.
3. Insert the legs into the frame. Adjust each leg to the desired height and ensure the push buttons click securely into place. Make sure all legs are set to the same height so the stool remains level.

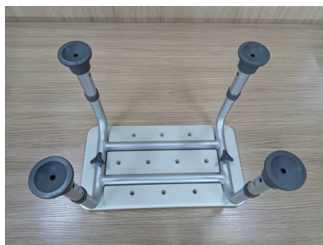

# Product Guidelines

---

1. Ensure all four legs are securely installed and evenly adjusted so the shower stool sits level and stable. Make sure there is enough space around the stool for safe movement.
2. DO NOT sit or place weight on only one side of the stool.
3. When sitting down or standing up, use a stable support such as a grab rail. Do not rely solely on the stool for balance.



## LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

# SectorCare



<https://sectorcare.com.au>