

Instruction Manual

Aluminium Standing Support Frame



SSFAL01WHI

SectorCare

Contents

Safety Instructions	1
Product Overview	2
Product Installation	3-5

Safety Instructions

When using the product, basic safety precautions should always be followed:

- ① Before use, check that the frame and armrest are not damaged or deformed, and ensure all screws are securely tightened.
- ② Use the product only on a flat, stable surface.



Product Overview

Congratulations on your purchase of our products. Our standing assist support are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



- 1 Padded Armrest Handle
- 2 Secondary Grip Handle
- 3 Height Adjustment Lock (Push-Button)
- 4 Lower Support Frame
- 5 Anti-slip Tips

Product Installation

1. Open the package and remove all components, as shown in Figure 1.



2. Install the lower support frame as shown in Figures 2 and 3.



3. Insert the bolt with washer through the aligned holes in the tube. Attach the nut with washer on the opposite side and tighten securely. Ensure all holes are properly aligned before tightening, as shown in Figures 4 and 5.



4. Insert the "H" frame into the lower support frame. Secure it using the C-shaped locking clamp, as shown in Figures 6 and 7.



5. Insert the top frame into the "H" frame. Secure it using the C-shaped locking clamp, as shown in Figures 8 and 9.



6. Check that the product is stable and all components are securely fastened.
The product is now ready for use



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>