

Instruction Manual

Aluminium Standing Support Frame



SSFAL01WHI

SectorCare

<https://sectorcare.com.au>



Product Overview



- 1 Padded Armrest Handle
- 2 Secondary Grip Handle
- 3 Height Adjustment Lock (Push-Button)
- 4 Lower Support Frame
- 5 Anti-slip Tips

Assembly Instructions



A-shaped round tube x1



H-shaped round tube x1



L-shaped round tube x1



Base x1



U-shaped clips x4



Nuts x4



Screws x4



Curved Gasket x8



Instruction Manual x1

Product Installation

1. Remove the product from the packaging.



2. Take out the L-shaped round tube and Base, align their mating joints and lay them flat together. Place Screws, Curved Gasket and Nuts in sequence over the joint and twist to tighten and secure, as illustrated:



3. Take out the H-shaped round tube and A-shaped round tube, align their mating joints and assemble them together, then secure with U-shaped clips, as illustrated:



4. Join the assembled components together and secure with U-shaped clips, as illustrated:



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.