

Instruction Manual

Toilet Aid with Suction Cup Caps



TSFAL01MSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



- 1 Handle
- 2 Handle Frame
- 3 Commode Fixing Brackets
- 4 Height-Adjustable Leg
- 5 Anti-slip Foot Pads

Assembly Instructions



Arm Main
Frame x2



Curved Round
Tubes x2



Leg Tubes x4



Commode Fixing
Brackets x2



U-Clips x2



V Nuts x2



Curved
Gasket x4



Round
Nuts x2



Black Plastic
Rings x2



Instruction
Manual x1

Product Installation

1. Remove the product from the packaging.




2. Align Curved Round Tubes with the connectors of Arm Main Frame and insert, secure with U-Clips, as illustrated:



3. Take out four Leg Tubes, align them with the connectors under Arm Main Frame and insert, as illustrated:



4. Take out Commode Fixing Brackets, insert them into the connectors on both sides of Arm Main Frame, then install and fix the position with Round Nuts, V Nuts and Curved Gasket, as illustrated:

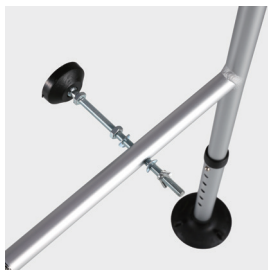

5. Finally, insert Black Plastic Rings into the end positions of Commode Fixing Brackets to secure, as illustrated:



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.