

Instruction Manual

Toilet Aid with Suction Cup Caps



TSFAL01MSIL

SectorCare

Contents

Safety Instructions	1
Product Overview	2
Product Installation	3-4
Product Guidelines	5

Safety Instructions

When using the product, basic safety precautions should always be followed:

- 1 The standalone toilet safety frame is designed to provide additional support and stability when using the toilet. Ensure the left and right frames are properly positioned and securely fixed before use. The frame must remain stable and should not move when sitting down or standing up.



Product Overview

Congratulations on your purchase of our products. Our walker are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



- 1 Plastic Handle
- 2 Handle Frame
- 3 Support Bracket
- 4 Height-Adjustable Leg
- 5 Horizontal Tubes

Product Installation

1. Unpack the product and check that all components match the parts listed in this manual.



2. Insert the leg extensions into the frame. Press the push button and slide the leg to the desired height. Ensure the button is fully engaged in the adjustment hole. Adjust all four legs to the same height to ensure stability and prevent rocking.



3. Insert the left and right frames into the two horizontal tubes. Secure them using the C-clips. The horizontal tubes can be adjusted to achieve the desired width.



4. Attach the holding kit to the left and right frames and secure it using the wing nuts. Position the toilet safety frame around the toilet. Ensure both sides of the holding kit are firmly attached and stable before use



Product Guidelines

Height Adjustment

1. Before use, adjust height of each leg to your specific needs, ensure all legs are at the same height. Once the desired height is reached, lock the legs into place by aligning the push buttons with the corresponding holes.
2. To ensure the legs are securely locked, gently push down or pull up on each leg to check stability.
3. Make sure the toilet safety frame is level before use.

Width Adjustment

1. Adjust the horizontal tubes to the desired width and secure it by C-clip.

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>