

Instruction Manual

Toilet Aid with Suction Cup Caps



TSFST01WHI

SectorCare

Contents

Safety Instructions	1
Product Overview	2
Product Installation	3-5
Product Guidelines	6

Safety Instructions

When using the product, basic safety precautions should always be followed:

- ❶ DO NOT exceed the maximum user weight limit.
- ❷ DO NOT fit any parts or accessories to your height-adjustable step manufactured by another company.
- ❸ DO NOT use if the rubber feet are damaged.



Product Overview

Congratulations on your purchase of our products. Our toilet safety frame are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



- 1 Plastic Handle
- 2 Plastic Toilet Clip
- 3 Adjustable Legs
- 4 Cross Support Frame
- 5 Non-Slip Rubber Feet

Product Installation

1. Carefully remove all packaging. All parts and accessory are shown in the picture below.



2. Insert the two adjustable horizontal tubes into the armrest frame. Secure the short tube using the provided C-shaped clips, as shown in the figure. Ensure all parts are properly aligned and firmly connected



3. Insert the short tube into the armrest on the opposite side. Use the two adjustable horizontal tubes to align the holes correctly. Secure them using the C-shaped clips, as shown in the figure.



4. Attach the nonslip rubber tips to the bottom of each leg. Ensure they are securely fitted before use.



5. Insert the long screws into the armrest frame from the outside and then install plastic toilet clip as shown in the picture.



6. Adjust the width of the frame to fit the toilet. Position it as close as possible to the toilet bowl, then tighten the toilet clamps securely.



Product Guidelines

Height Adjustment

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. Make sure all legs are locked in place firmly, check with pushing down the frame gently.
3. Make sure that the toilet safety frame is leveled.

Width Adjustment

1. Adjust the horizontal tubes to the desired width and secure it by C-clip.

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>